

It's as easy as **1, 2, 3...**

1. Fill

Simply lift the green lid and pour in the medication. Close the lid and make sure your child is sitting up straight, ready for their therapy.



2. Treat

Press the on button and encourage your child to breathe in and out through the mouthpiece.** InnoSpire Go will indicate therapy completion with audible and visual signals – before turning itself off.




3. Clean

After each treatment remove the mouthpiece and rinse it in warm water. At the end of each day wash it in warm soapy water and once a week sterilize it in boiling water.



Find out more
www.philips.com/innospirego

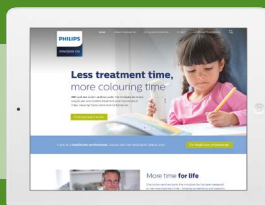

* Using 2.5ml salbutamol

** A mask is included for children that find it difficult to use the mouthpiece.



© 2016 Koninklijke Philips N.V. All rights reserved. Specifications are subject to change without notice. Respireonics Inc. and Respireonics Respiratory Drug Delivery (UK) Ltd are Philips companies.

Respireonics Respiratory
Drug Delivery (UK) Ltd
Chichester Business Park
City Fields Way
Tangmere
Chichester
PO20 2FT UK

KW xx/xx/2016 PN452287
www.philips.com/respironics

PHILIPS

InnoSpire Go

Portable mesh nebulizer



More time for life

More time for the yummy things in life

Fast, effective and easy to use – the InnoSpire Go nebulizer is designed to minimize asthma treatment time so your child can get back to doing something fun.

Treatment up to **25% quicker*****



Treatment in 4 mins**

An efficient asthma therapy device that indicates therapy completion using audible and visual signals – before automatically turning off.



Virtually silent

A small, portable and discreet nebulizer that can be used anywhere.



Built-in long life battery


The rechargeable battery provides up to 30 treatments between each charge.



Simple, only two parts

The easily detachable mouthpiece makes the InnoSpire Go simple to use, clean and maintain.

Find out more
www.philips.com/innospirego

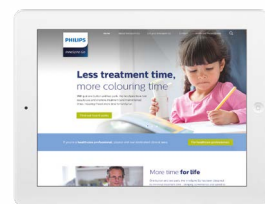

← Actual size →

* Using 2.5ml salbutamol.

** A mask is included for children that find it difficult to use the mouthpiece.

*** Versus predecessor range. Internal testing comparing Aeroneb Go.